


Pantera™ Pro

Semi-compliant
balloon catheter

High push. High cross coronary balloon.

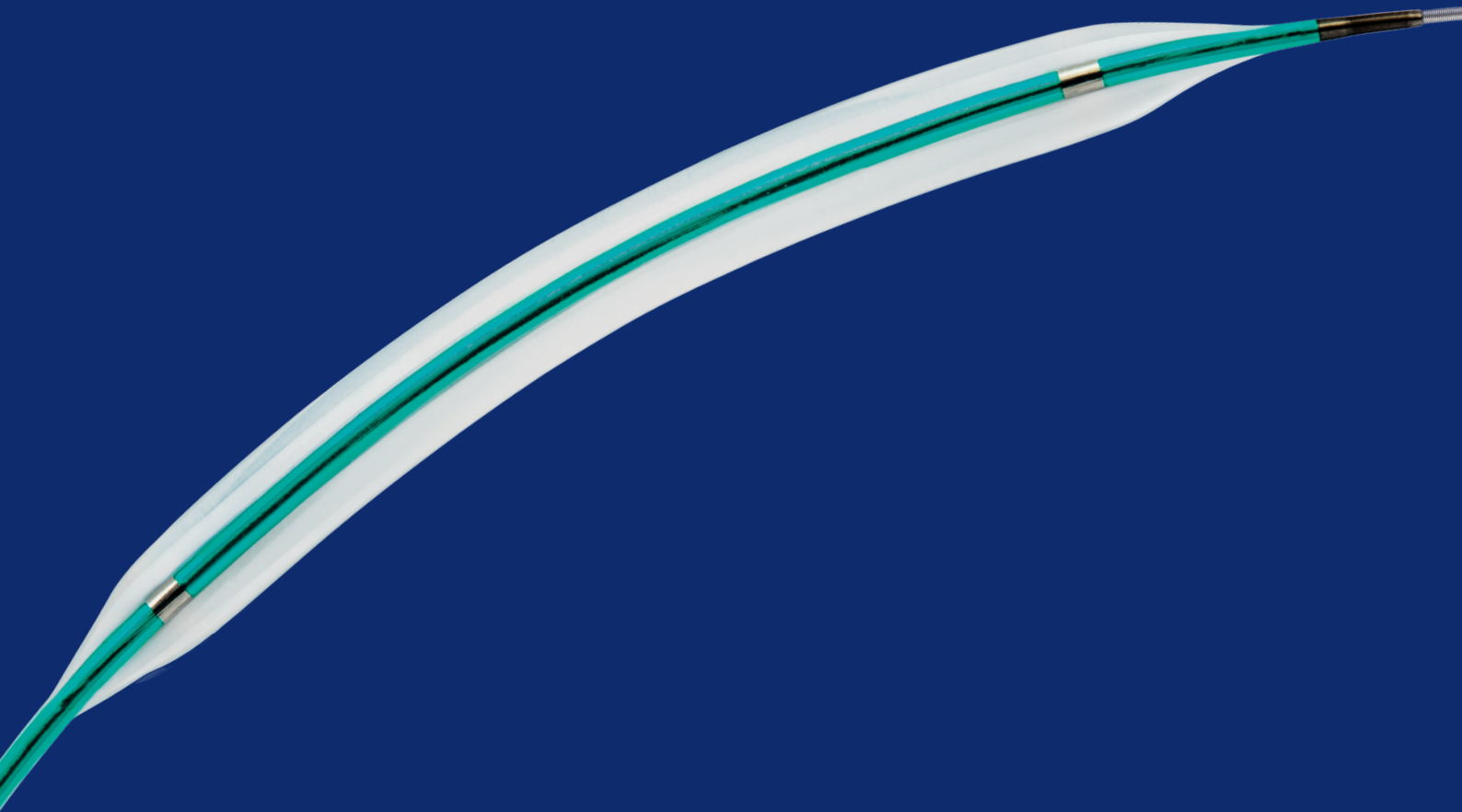


Pantera™ Pro

High push. High cross coronary balloon.

Better crossability¹

Slim shoulders and hydrophilic coating*

Proprietary balloon material for small sizes allows for slim shoulders while maintaining durability. Slim shoulders coupled with hydrophilic coating* facilitate better crossing.¹

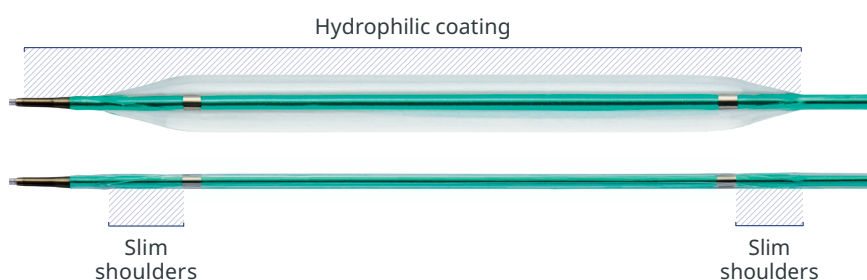


43 % less friction² during simultaneous use

Reduced distal shaft profile




The reduced distal shaft profile lowers friction when using two balloons in a 6F guiding catheter**

Lowest friction during simultaneous use of two balloons compared to main competitors²

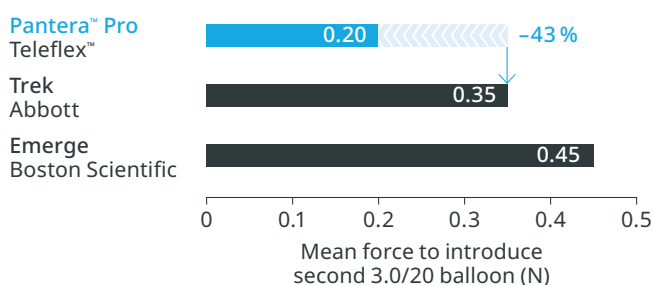

* $\varnothing$ 1.25–2.0 mm

** Any combination of two diameters not larger than 3.5 mm within a 6F guiding catheter with a minimal inner diameter of 0.070"/1.78 mm.

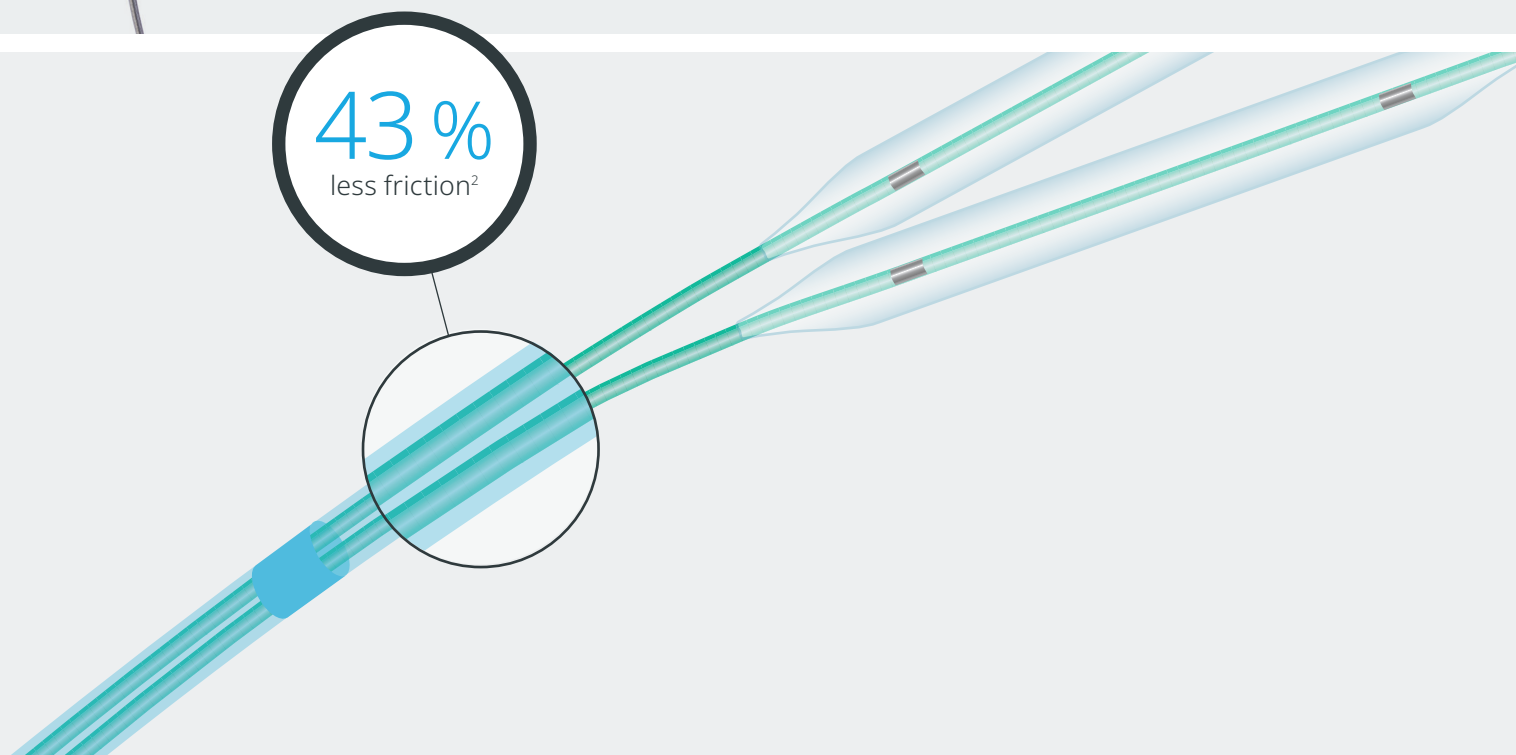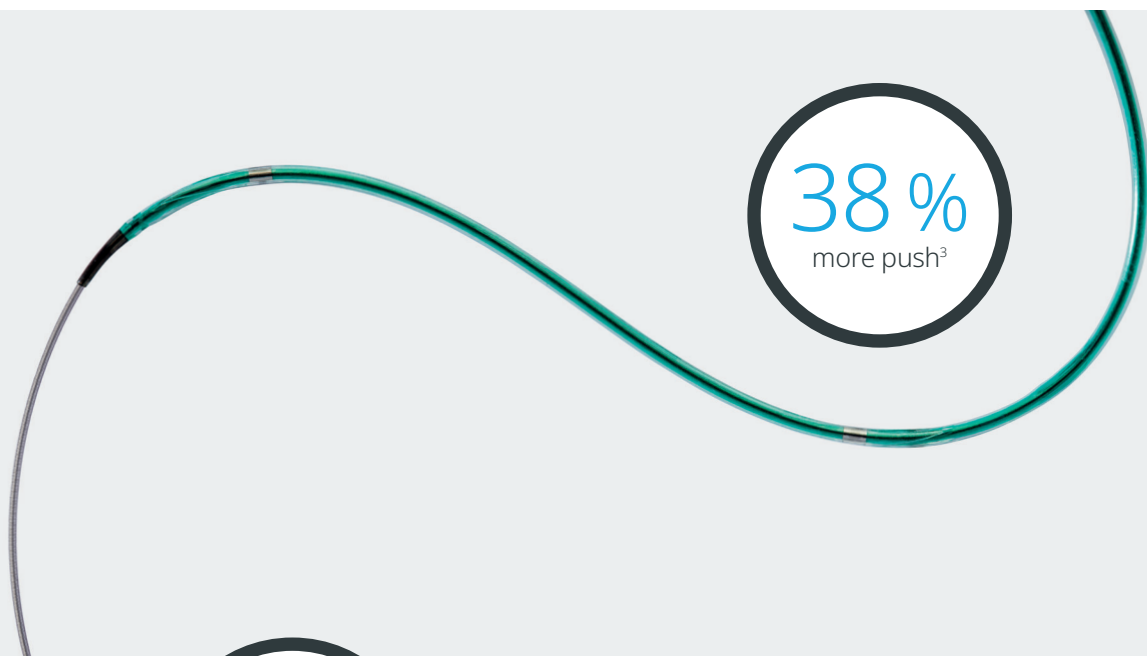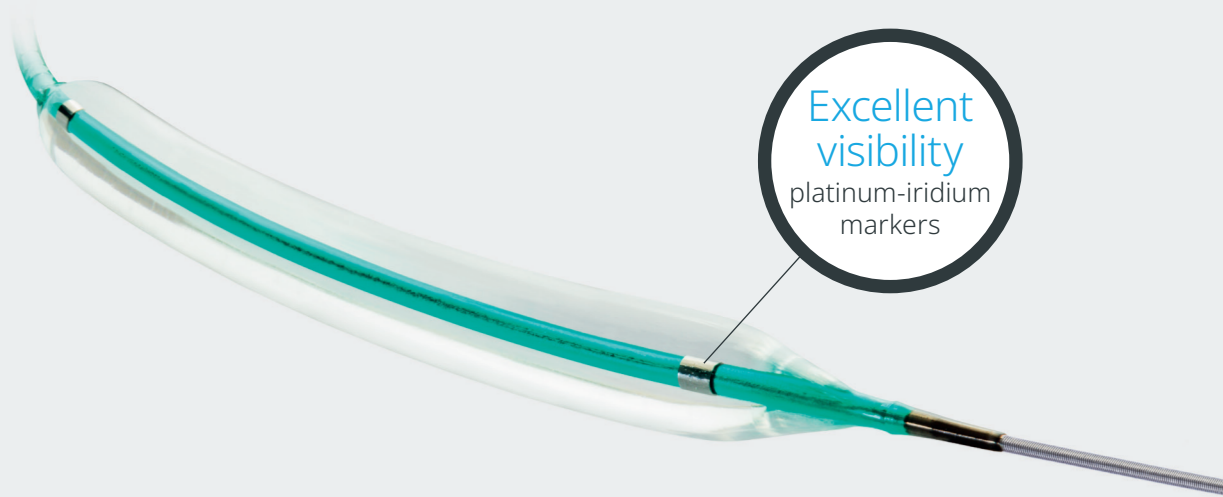


38 % more pushability³


Enhanced Force Transmission shaft

Teleflex’s unique Enhanced Force Transmission shaft results in optimal pushability due to the direct transition from proximal metallic hypotube to distal guide wire support section.


Pantera™ Pro

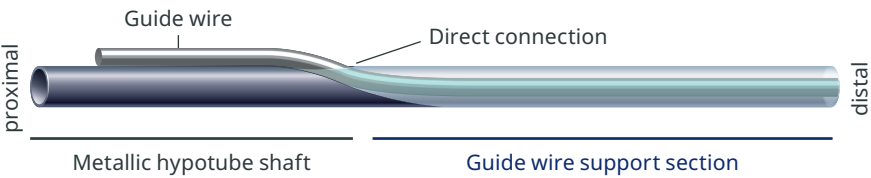


Competitors

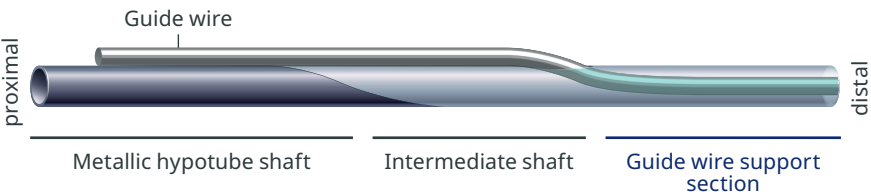

Pushability comparison

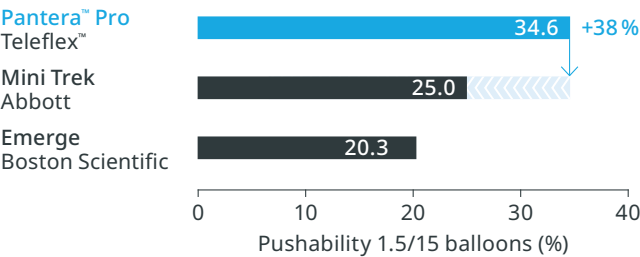

Pantera™ Pro

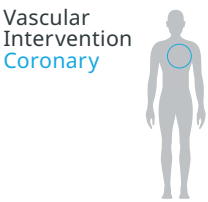

Indicated for dilatation of coronary artery or bypass graft stenosis.*

Technical data

PROXIMAL SHAFT	
Design	Hypotube design
Diameter	2.0F
Shaft markers	92 cm and 102 cm from tip
DISTAL SHAFT	
Guiding catheter	5F (min. I.D. 0.056" / 1.42 mm)
Guide wire diameter	0.014"
Lesion entry profile	0.017"
Usable length	140 cm
Balloon material	Semi Crystalline Co-Polymer
Balloon folding	ø 1.25–1.5 mm: Two-fold; ø 2.0–4.0 mm: Tri-fold
Balloon markers	Platinum-Iridium: ø 1.25–1.5 mm one marker; ø 2.0–4.0 mm two markers
Coating distal shaft	Hydrophilic (end of balloon to Guide Wire (GW) exit port)
Balloon and tip coating	ø 1.25–2.0 mm: Hydrophilic ø 2.50–4.0 mm: Hydrophobic
Simultaneous use compatibility	6F guiding catheter (min. I.D. 0.070" / 1.78 mm), up to ø 3.5 mm
Diameter	2.6F (ø 1.25–2.0 mm); 2.7F (ø 2.5–3.5 mm); 2.9F (ø 4.0 mm)

Compliance Chart

		BALLOON DIAMETER × LENGTH (MM)						
		ø 1.25 × 6–20	ø 1.50 × 6–20	ø 2.00 × 10–30	ø 2.50 × 10–30	ø 3.00 × 10–30	ø 3.50 × 10–30	ø 4.00 × 10–30
Nominal Pressure (NP)	atm**	7	7	7	7	7	7	7
	ø (mm)	1.24	1.49	2.01	2.49	3.08	3.62	3.95
Rated Burst Pressure (RBP)	atm**	14	14	14	14	14	14	14
	ø (mm)	1.37	1.72	2.23	2.93	3.50	4.06	4.55

**1 atm = 1.013 bar

Ordering Information

BALLOON Ø		CATHETER LENGTH 140 CM, BALLOON LENGTH					
		6 mm	10 mm	15 mm	20 mm	25 mm	30 mm
5F	1.25 mm	393289	393291	393298	393305	–	–
	1.50 mm	393290	393292	393299	393306	–	–
	2.00 mm	–	393293	393300	393307	393312	393317
	2.50 mm	–	393294	393301	393308	393313	393318
	3.00 mm	–	393295	393302	393309	393314	393319
	3.50 mm	–	393296	393303	393310	393315	393320
	4.00 mm	–	393297	393304	393311	393316	393321

*Indication as per IFU

References

- 1 Bench test with 1.5/15 mm size when compared to Mini Trek (Abbott) and Emerge (Boston Scientific), data on file.
- 2 Bench test with 3.0/20 mm size when compared to Trek (Abbott), data on file.
- 3 Bench test with 1.5/15 mm size when compared to Mini Trek (Abbott), data on file.

Refer to IFU

Refer to QR code for a copy of the Instructions for Use for a complete listing of the indications, contraindications, warnings and precautions.



About Teleflex

As a global provider of medical technologies, Teleflex is driven by our purpose to improve the health and quality of people's lives. Through our vision to become the most trusted partner in healthcare, we offer a diverse portfolio with solutions in the therapy areas of anesthesia, emergency medicine, interventional cardiology and radiology, surgical, vascular access, and urology. We believe that the potential of great people, purpose driven innovation, and world-class products can shape the future direction of healthcare.

Teleflex is the home of Arrow™, Barrigel™, Deknatel™, LMA™, Pilling™, QuikClot™, Rüsch™, UroLift™ and Weck™ – trusted brands united by a common sense of purpose.

At Teleflex, we are empowering the future of healthcare. For more information, please visit teleflex.com.

Corporate Office

Phone +1 610 225 6800, 550 E. Swedesford Road, Suite 400, Wayne, PA 19087, USA

Regional Offices

United States: Phone +1 919 544 8000, Toll Free 866 246 6990, cs@teleflex.com, 3015 Carrington Mill Boulevard, Morrisville, NC 27560, USA

Latin America: Phone +1 919 433 4999, la.cs@teleflex.com, 3015 Carrington Mill Boulevard, Morrisville, NC 27560, USA

International: Phone +353 (0)9 06 46 08 00, orders.intl@teleflex.com, Teleflex Medical Europe Ltd., IDA Business and Technology Park, Dublin Road, Athlone, Co Westmeath, Ireland

Teleflex, the Teleflex logo, and Pantera are trademarks or registered trademarks of Teleflex Incorporated or its affiliates in the U.S. and/or other countries. All other names are the trademarks or registered trademarks of their respective owners.

Information in this material is not a substitute for the product Instructions for Use. Not all products may be available in all countries. Please contact your local representative.

© 2025 Teleflex Incorporated. All rights reserved.

413275-EN · REV E · 09 25 PDF